

DAYS OF THE WEEK BANDAGES!

Help clients and staff remember when a dressing was applied or when a dressing needs to be changed with...

Andover's **DAYS OF THE WEEK PETFLEX® BANDAGE!**



Applying Andover's Days of the Week PetFlex Bandage



Step 1: Wrap with PetFlex, cohesive flexible bandage.



Step 2: Wrap one layer around top of PetFlex (it will stick to the bottom layer). Circle the day with a sharpie pen (either the day it was applied or the day it needs to be changed).*

*Protocol to be determined by clinic/hospital.

Other Application Ideas:

- Circle the day the animal needs to come in for a follow-up visit as an easy reminder for the client.
- Circle the day a catheter was applied or the day it needs to be removed. Bandage has enough space to write a time, if needed.
- Circle the day the animal will be discharged as an easy reminder for staff.
- Circle the day the dressing needs to be changed when an animal is sent home after surgery.

ORDERING INFORMATION

Catalog #	Size	Rolls/Box	Boxes/Case
2200DW-024	2" x 5 yds.	6	4



EasyTear
PATENTED TECHNOLOGY



Andover Healthcare, Inc.

9 Fanaras Drive, Salisbury, MA 01952 Toll Free: 800.432.6686 Phone: 978.465.0044 Fax: 978.462.0003

www.andoverhealthcare.com

Proudly Made in USA

DOW080509